
ENGLISH : Single-use filtering face-piece respirators
DEUTSCH : Filtrierende Halbmasken
FRANÇAIS : Masques respiratoires a usage unique
NEDERLANDS : Filtrenderde halfmaskers
ITALIANO : Respiratore facciale filtrante monouso
ESPAÑOL : Mascarillas autofiltrantes desechables
PORTUGUÊS : Respiradores filtrantes
SVENSKA : Filtrenderde andningsskydd
DANSK : Filtrende ansigtsmaske
NORSK : Filtrende masker
SUOMEKSI : Kertakäyttöinen suodattava hengityssuojain
MAGYAR : Egyszer használatos, szűrős légző-arcmaszk
ČECH : Filtrové lícní respirátory na jedno použití
POLSKI : Maski ochronne jednorazowego użytku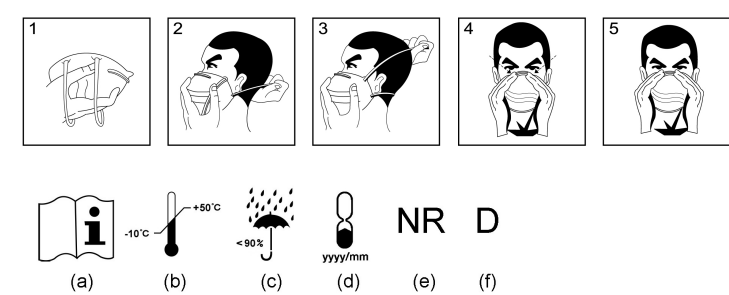

CE 2797
BSI Group The Netherlands B.V.
Say Building, John M. Keynesplein 9, 1066 EP
Amsterdam, Netherlands

AMSTERDAM, NETHERLANDS

ENGLISH : INSTRUCTIONS FOR USE

IMPORTANT : EN 149:2001+A1:2009 single-use respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

APPLICATIONS : • **FFP1 :** protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4 x MAC/OEL/TLV, [i.e. NPF=4] or 4 x WEL, [i.e. APF=4]. • **FFP2 :** protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEL/TLV, [i.e. NPF=12] or 10 x WEL, [i.e. APF=10]. • **FFP3 :** protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEL/TLV, [i.e. NPF=50] or 20 x WEL, [i.e. APF=20]. ***NOTE :** MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. NPF = Nominal Protection Factor. WEL = Workplace Exposure Limit. APF = Assigned Protection Factor.

CHECK PRIOR TO USE : Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, ***DO NOT USE IT***. Get a new one.

FITTING INSTRUCTIONS : **1.** Hold the respirator in hand with the nosepiece at your fingertips. Allow headbands to hang freely below hand. **2.** Cup the respirator firmly against your face with the nosepiece on the bridge of your nose. **3.** Stretch and pull the lower headband over the head and position below your ears. Stretch and pull the top headband on the back of your head above your ears. ****Respirators with adjustable headbands only :*** *while maintaining the position of the respirator, adjust tension by pulling tab of each headband. (Tension may be decreased by pushing out the back of the buckle.)* **4.** Press soft nosepiece to conform snugly around the nose. **5.** (a) To test fit for respirator without exhalation valve : Cup both hands over the respirator and exhale vigorously. (b) To test fit for respirator with exhalation valve : Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit. ***NOTE :** Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observance of these instructions is an important step in safe respirator use.

WARNING : **1.** Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death. **2.** A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use. **3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%. **4.** Do not use when concentrations of contaminants are immediately dangerous to health or life. **5.** Do not use in explosive atmospheres. **6.** Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs. **7.** Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator. **8.** Never alter or modify this respirator in any way. **9.** Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use. **10.** Keep respirators in the display box away from direct sunlight until use.

Manufactured/Imported by : Refer to packaging for contact details.

MEANING OF PICTOGRAMS/SYMBOLS : (a) See user information. (b) Temperature range of storage condition. (c) Maximum relative humidity of storage condition. (d) End of shelf life. yyyy = Year; mm = Month. (e) NR : this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used. (f) D : this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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